

Frequently Asked Questions



Why is the elastic in the Tube Holders?

This is a safety device which allows the patient to cough without coughing out the Tracheostomy Tube. The elastic stretches during coughing

Can I wash the Holder?

The Tracheostomy Tube Holder is only manufactured as a single use product

How can I obtain the product in the UK

It is available on prescription and through your Pharmacist

The Saint Marina Safety Net: What is the purpose of the product

It has two main uses. Firstly, it is cosmetically more acceptable than just an open Tracheostomy stoma and it also acts to keep the air passing to the trachea and lungs warm and humid

T566 Endotracheal Tube Holder: How can I adjust the size of the product to use with different neck sizes?

Wrap the part that fits onto the Endotracheal Tube around the Endotracheal Tube after fixing, as many times as necessary to meet the patients neck size, before attaching to the velcro on the neck band.

Wet Protect: When should the Wet Protect be used?

During showering, shaving and hair trimming to stop anything going onto the tracheostomy stoma